CLASSIDOO

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- F** Notice d'utilisation
- EN** Instructions
- D** Gebrauchsanweisung
- E** Folleto de Instrucciones

KMF SARL
51, route de Bièvres
91370 Verrières le buisson
FRANCE
MADE IN HUNGARY

www.tomyka.fr
Tél. : 01 60 19 47 39
info@tomyka.fr

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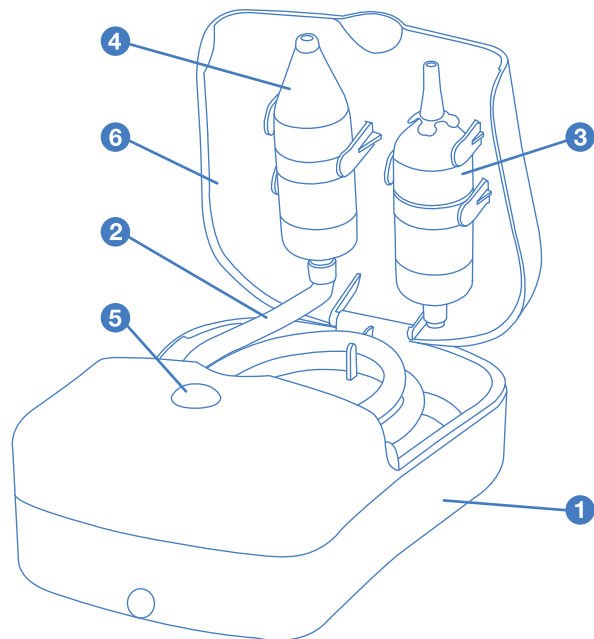
PLEASE READ THIS INSTRUCTION LEAFLET CAREFULLY
AND KEEP IT IN A SAFE PLACE.

- 1 Vacuum pump.
- 2 Silicone pipe.
- 3 Tip from birth onwards.
- 4 Tip for 3 months upward.
- 5 Switch.
- 6 Storage case.

EN

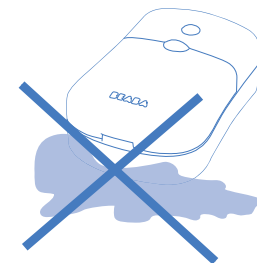
We thank and congratulate you on your choice : you have purchased
an appliance that has unique performance features.

TECHNICAL FEATURES

- Requires two AA-LR6 batteries or an adaptor.
- Type SM-89-03.
- Vacuum pump size : 145 x 99 x 56 mm.
- Net weight : 250 g.
- Power supply : U1 : 230-V50 Hz power 7 W
U2 : 120-V50 Hz power 6,5 W.
- Transformer : AC : 230/DC 3V 500 mA
AC : 120/DC 3V 500 mA.
- Model : D35-03-500 / Double isolation / Type B /
- Grade of protection : IP 20
- Storage temperature : 0-40°C
- Vacuum : -0,4 bar.

WARNING

Never immerse the electrical pedestal
unit in water.

USING THE APPLIANCE

- Put the unit on a dry, non slippery level surface (eg. towel).
- Open the lid of the baby nose cleaner and select the tip adapted to the age group.
- Select the right nose-cleaning nozzle customized for your kid's age.
- Comes with 2 anatomically-shaped tips:
 - 1 tip  : from birth onwards,
 - 1 tip  : for 3 months upward.
- Position the nozzle on the entry of the obstructed nostril, without fletting the air return.
- To optimise the nose-cleaning, it is recommended to close the reverse nostril with your finger.
- Do not use the apparatus for more than 3 mn. Wait 1 mn before using again.
- you can use it as often as necessary. A few drops of saline water can be used in order to aid blowing the nose: the drops will act as a decongestant.

CLEANING INSTRUCTIONS

- Always unplug the nose-cleaner before cleaning.
- Drop the nose-cleaning nozzle.
- After every nose-cleaning, the nozzle should be cleaned with cold water or sterilized. If secretions go into the silicone pipe, dis-assembly it from the unit, clean and dry it before re-use.
- If secretions are found in the device, clean it following the instructions on the diagram below:
- Pour some warm water into a glass.
- Put the silicone tube in the glass of water.
- Position your device over a container or the sink.
- Switch it on.
- The product may be rinsed several times.

SAFETY RECOMMENDATIONS

- Always unplug the nose-cleaner after each use.
- Never immerse the unit into water or under running water.
- Clean the motor base (A) with a damp cloth only. Never use abrasive cleaning products.
- Do not use the appliance with a damaged cord, or if unit is broken.
- Close supervision is necessary when appliance is used near children.

GUARANTEE

- In addition to the legal warranty provided for by law on hidden defects, in any event, KMF sarl guarantees this product exclusively for domestic usage against any manufacturing fault or material defect.
- The warranty period is 1 year from the date of purchase. During this period, the product will either be repaired or replaced free of charge.
- The guarantee is only valid if proof of purchase is produced with the product.
- It does not cover damage resulting from any use of the product not described in the instructions or of the product is knocked or dropped.
- KMF sarl will not be held responsible for material damage or personal accidents resulting from any installation that does not comply with legal or statutory provisions.
- Calling a guarantee does not give right to damages and interest.
- We reserve the right to implement any modifications to the technical, design or colour specifications that we see fit to do.
- This document has no contractual value apart from its guarantee clauses.

WARNING

- Danger avoidance: if the power cable is damaged it must be replaced by our after-sales service.
- Tel. +33 (0)1 60 19 47 39
- email address : info@tomyka.fr

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