Cachet / garantie 2 ans

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USER INSTRUCTIONS

Inserting the batteries (diagram A):

Open the battery compartment **(10)** while separating it from the body of the baby nasal aspirator **(1)**, and insert the 2 x LR06 batteries, the right way round, then close the compartment.

Choice of nozzles:

the baby nasal aspirator is supplied with 2 nozzles, one for very small infants (0-3 months), for tiny nostrils, and a larger nozzle for bigger babies.

Starting by pressing the electronic switch:

one press: orange light for slight nasal congestion.

Two presses: blue light for heavier nasal congestion.

When the battery level is too low, the **blue light** flashes.

Aspiration of secretions: place the silicone nozzle **(3)** at the entry of the nostril leaving no air gap between the nostril and the nozzle. Draw in for a few seconds; the secretions are recovered in the cup **(6)**. Be careful not to draw in against the wall of the nostril. You can repeat the operation as many times as necessary.

Separate the terminal section (diagram B):

Pull the silicone nozzle **(3)**, unscrew the transparent cup **(4)**, unscrew the cup **(6)** to separate it from the body of the aspirator **(1)**, check that the silicone seal is correctly positioned **(5)**, screw in and assemble the various parts.

Cleaning after use: Terminal part: separate the terminal part from the baby nasal aspirator **(diagram A and B)** and rinse the various components in cold water. They can be sterilized cold. Do not sterilize in a microwave oven.

Technical description:

aspiration power: level 1 : 47 kpa / level 2 : 54 kpa

sound level: level 1 : 62 dB / level 1 : 65 dB

Delivered with: 2 x LR6 – AA 1.5 V batteries, changeable nozzles to suit the size of the nostril, replacement silicone seal, transport wallet and hygienic cap.

USER ADVICE

To improve the efficiency of the product, it is recommended to hold the other nostril closed during aspiration. It only takes a few seconds to draw off the secretions. Do not use the baby nasal aspirator if the nostrils are irritated. It is advisable to rinse the terminal section with cold water after each aspiration. Keep the product in a dry place at ambient temperature. If it is not used for several months, it is advisable to remove the batteries. The Tomydoo baby nasal aspirator helps prevent respiratory complications; in case of infection, it is better to see a doctor.

PRECAUTIONS

Tomydoo is designed for the aspiration of nasal secretions of infants. Do not use in any other circumstances. Before use, ensure that the various components are properly assembled. Do not push the nasal nozzle into your baby's nostril. Never leave your Ballidoo within reach of children. Do not clean the device with the battery compartment open. Do not unscrew the screws or seals as this could damage the appliance.

Do not clean with solvents or abrasives.

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